

Application of heat mount detectors



1

Apply thin layer of glue to patch and set aside to become tacky—Some detectors come pre-glued and do not require this step.

Trim long hair to 1-2 cms if needed so detector does not shift.



2

Remove loose dirt and hair from cows tailheads.



3

Ideal placement is for the edge of the patch to be parallel to hip bones mounted centered on the back bone.

Apply glue to selected area ensuring hair penetration.

Detector should be placed on smooth part of spine, not in a depression.

Allow time for glue on cow to become tacky.



4

Apply patch, arrows point forward to prepared area.

Apply firm pressure to side panels of patch for good bond.

Always refer to manufacturer's instructions for unique application requirements. Use the recommended adhesive.